



SPACER S PREPARATION STAGES

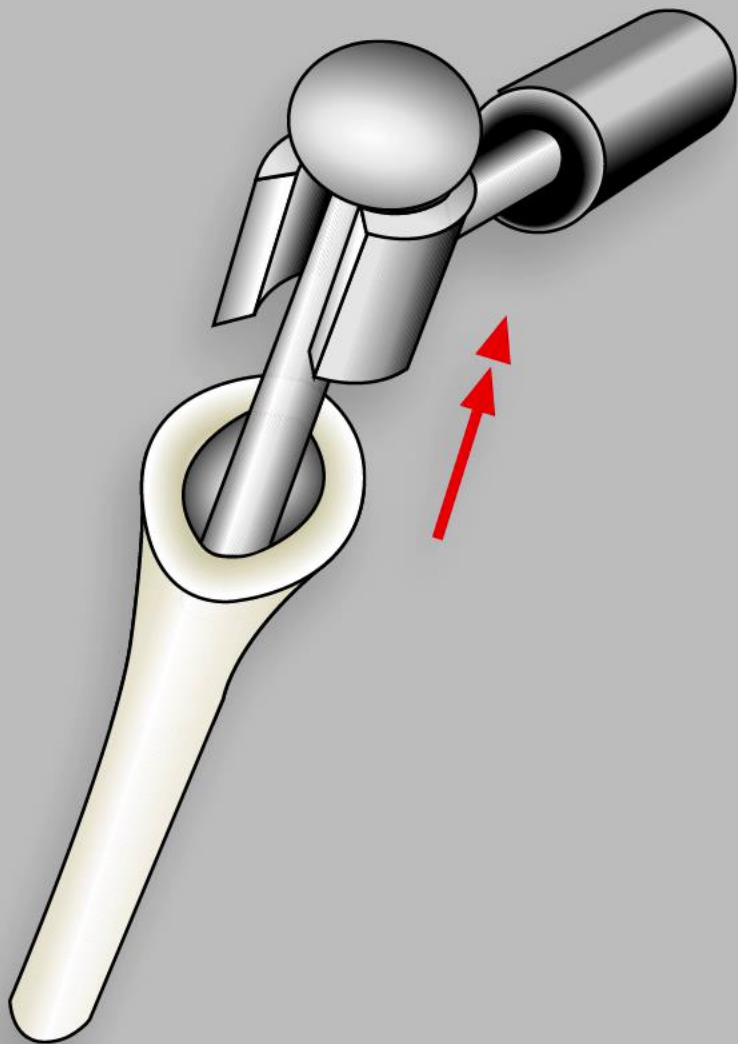

**REMOVE THE PROSTHESIS
AND CLEAN THE HOST SITE BY
AGGRESSIVE DEBRIDEMENT,
REMOVING ALL RESIDUES
OF CEMENT, IF PRESENT**

SPACER S PREPARATION STAGES

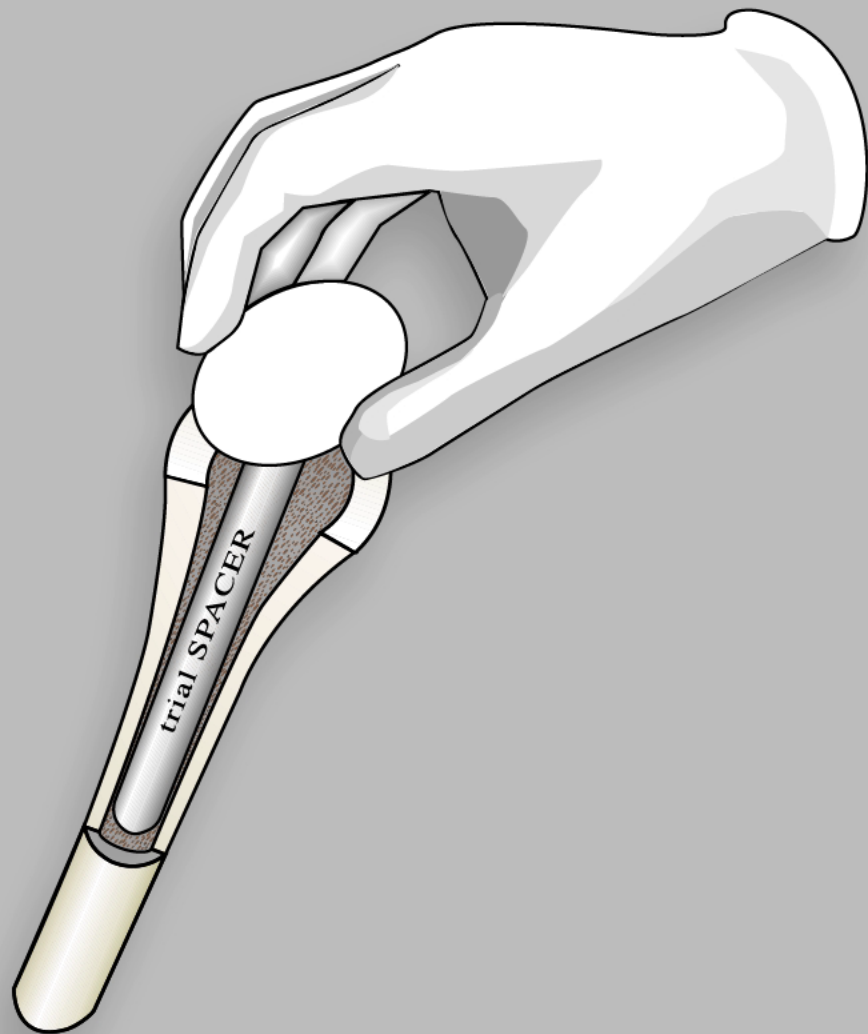


**SELECT THE RIGHT SIZE
WITH THE TRIAL DEVICE**




TECRES
ADVANCING HIGH TECHNOLOGY

SPACER S PREPARATION STAGES

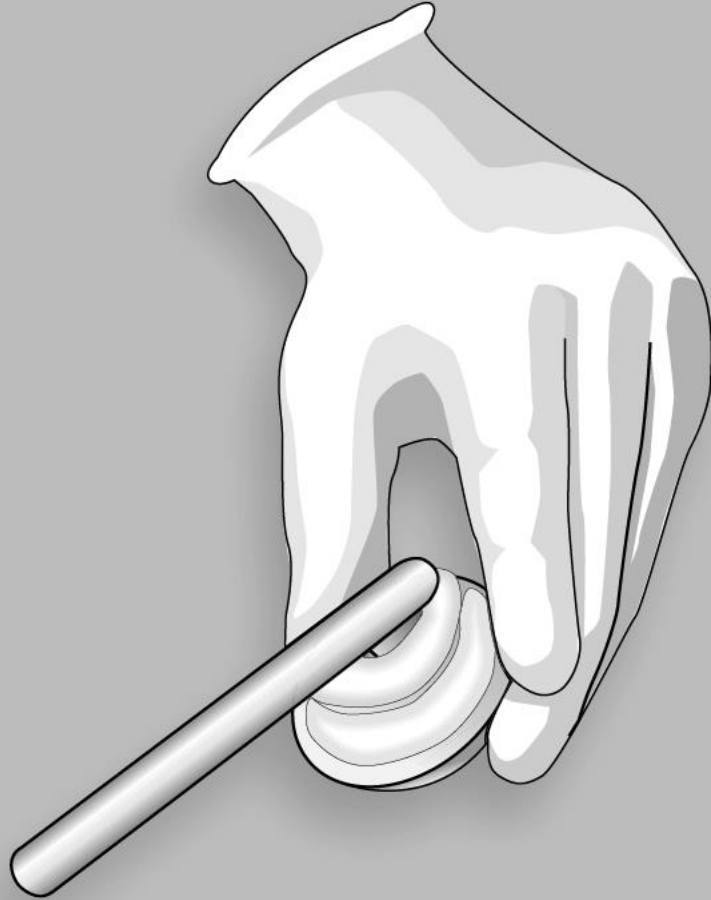


**APPLY ANTIBIOTIC
LOADED BONE CEMENT
TO UNDERSIDE
OF SPACER HEAD**

SPACER S PREPARATION STAGES

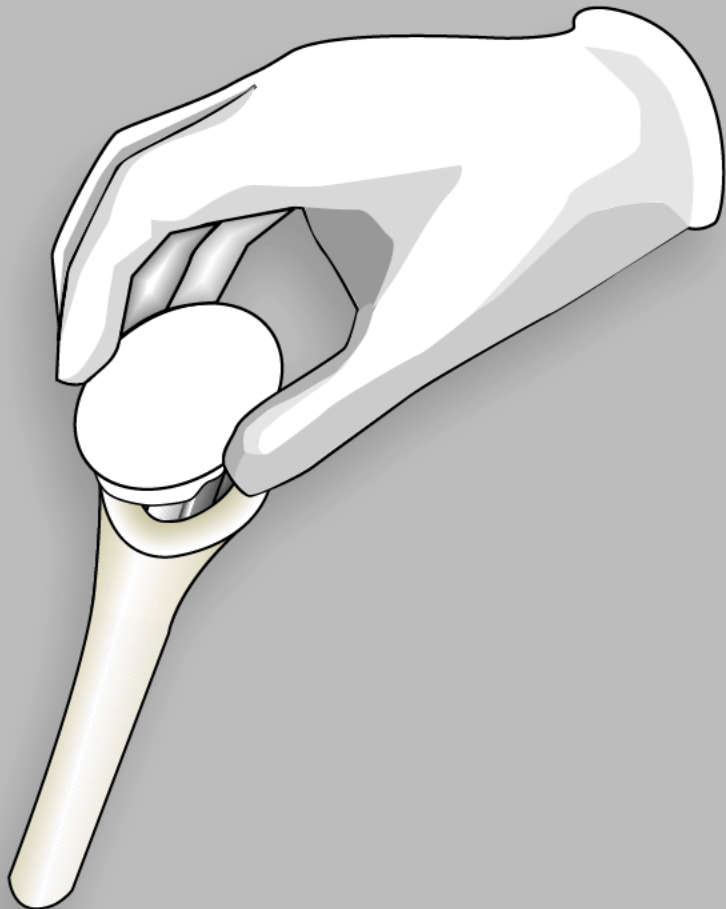

**INSERT THE SHOULDER SPACER
INTO THE HUMERAL CANAL
WHILE APPROXIMATING
ANATOMICAL VERSION**

SPACER S PREPARATION STAGES

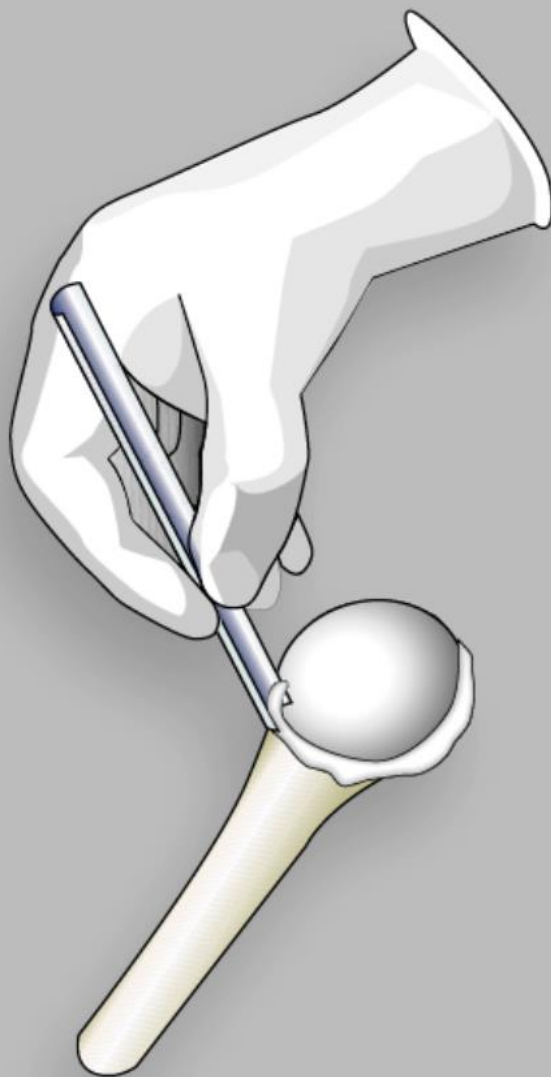

**REMOVE ALL EXTRUDING BONE
CEMENT WITH A CURETTE
AND REDUCE THE SHOULDER
(OPTIONAL)**

SPACER S PREPARATION STAGES

**ACCORDING TO THE STABILITY ACHIEVED,
AND THE ANATOMICAL CONDITION,
PARTIAL OR TOTAL MOBILITY
WILL BE ALLOWED BY THE SURGEON.
AN ORTHOPADIC BRACE MAY BE APPLIED**